


ClearViewTM

Anti-Beschlag Optik Wärmer

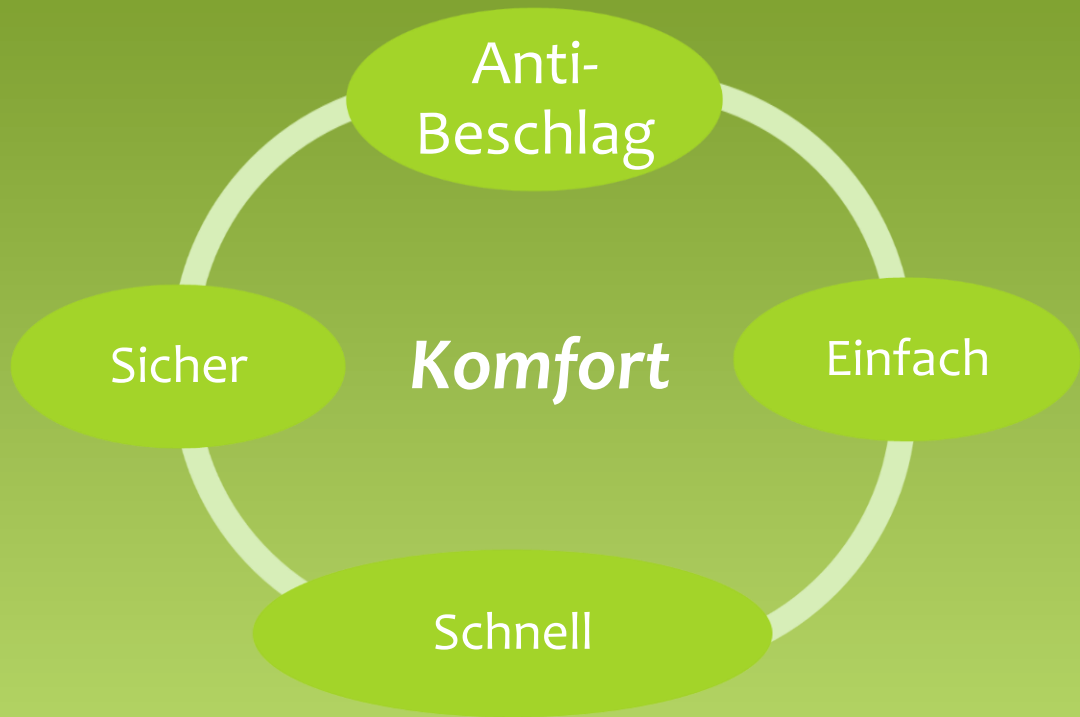

Heizt der Optik auf Körpertemperatur vor

Kein Strom oder heißes Wasser

Aktiviert durch einfaches Anklicken



supporting surgical outcomes


ClearViewTM

Anti-Beschlag Optik Wärmer




Instructions for use

1. OPTIK(EN) IN WÄRMER LEGEN

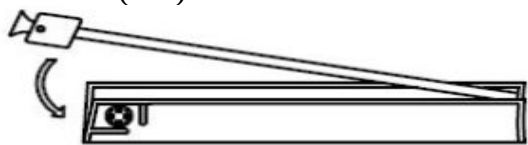

2. SCHEIBE BIEGEN

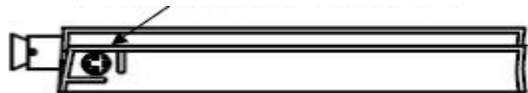

3. WARM FÜR DEN GEBRAUCH


Achtung: Der Optikwärmer sollte 15 Minuten vor Gebrauch der Optik aktiviert werden.

ClearViewTM bleibt bis zu 2 Stunden warm.

Abdecken mit sterilem Tuch um Wärme zu erhalten.

References

Clearview scopewarmer generates a significant higher difference during the complete process of pre-warming the endoscope. Clearview exchanges 75% more energy at the start compared to a warm water bath. Over time the diffusion capacity of a warm bath reduces to only 12% of the heat flux from the Clearview scopewarmer*. This significant difference ensures that enough energy is diffused during the process to ensure the whole endoscope, including the optic materials inside, are pre-warmed to body temperature.

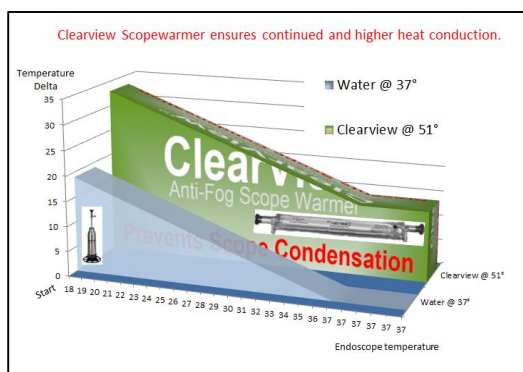

* Data on file

supporting surgical outcomes